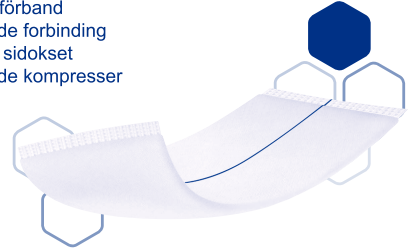
Heliderm®

Absorbent dressings
Абсорбиращи компреси
Upijajuće obloge
Comprese assorbente
Vpojne obloge
Absorptionsförband
Absorberende forbinding
Imukykyiset sidokset
Absorberende kompresser
吸水性敷料
伤口敷贴
بطانة ماصة
پانسمان جاذب

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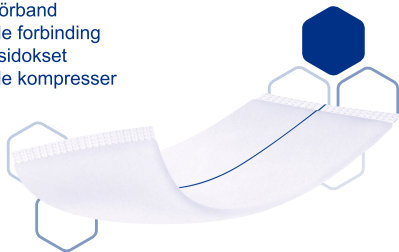
Product description:

Heliderm is a Sterile, Single-Use Absorbent Dressing used on superficial, moderately to highly exuding wounds. The absorbent core absorbs and binds exudate.

Sterile, Single-Use Absorbent Dressing for long-term treatment of injured skin, acute and chronic, with moderate to high levels of exudate. It is used on adults only, by healthcare professionals in clinical or homecare environments. It can be combined with local antiseptics, primary and secondary dressings and used under compression bandages.

Composition:

Heliderm is a combined absorbent dressing which consists of four layers of different materials. On the wound side, the product features a soft, white, hydrophilic non-woven. The inner dressing core consists of soft cellulose fluff blended with liquid-retaining polyacrylate polymers. This absorbent core is enclosed in a thin nonwoven fabric that evenly distributes the liquid. On the side facing away from the wound, the product features a green layer of polypropylene nonwoven, which is water-repellent but permeable to air and allows for gas exchange.

A blue indicator strip is integrated on the back of the dressing to facilitate easy handling and application. The dressing also offers superior softness and excellent conformability, ensuring an optimal anatomical fit over the wound and enhancing overall patient comfort.

Properties and mode of action:

rapidly absorbs exudate and binds it within the absorbent core. Exudate removal eliminates inhibitory factors from the wound, e.g. proteases. The increased absorption capacity of Heliderm reduces the required frequency of dressing changes. This promotes wound rest and provides additional protection against contamination. Apart from its absorbent quality, also has a padding effect.

Indications for Use:

Heliderm is indicated for the management of superficial wounds with moderate to high exudate levels, including acute wounds (traumatic wounds, post-operative wounds, lymphatic wounds) and chronic wounds (pressure ulcers, venous, arterial or mixed leg ulcers, and tumour wounds). It is also suitable for use under compression bandages.

Mode of application and Precautions:

- Select Heliderm to match the wound size so that the dressing extends at least 1-2 centimeters beyond the wound margins.
- Place the white side of Heliderm on the wound so that the blue

indicator strip layer faces away from the wound.

- Secure the dressing e.g. with adhesive tape, conforming bandages or, if necessary, compression bandages.
- One dressing can remain on the wound for up to 7 days, depending on the condition of the wound. Change the dressing if clinically indicated or when exudate reaches the edges of the dressing or is noticeable through nonwoven top layer.

Contraindications

Do not use Heliderm on dry wounds or on exposed bones, muscles or tendons. Do not use in case of hypersensitivity to any of its components.

Warning:

- Do not cut the dressing.
- Medical assessment of the wound condition and the causes of wound-healing impairment is necessary before treating wounds with an impaired healing tendency.
- Treatment with Heliderm cannot replace a causal treatment of the wound-healing impairment. If there are clinical signs of infection, the infection needs to be controlled with appropriate treatment before this dressing can be used.

- In all cases, follow the established clinical protocol.

In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution and following a clinician's recommendation.

- For dressings which are packed in paper-paper peel packs: The sealed seam of the peel pack contains natural rubber latex, which may cause allergic reactions!

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

- Before opening the product packaging, inspect it carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.

Storage conditions:

Keep in a dry place and protect from moisture.

Keep away from direct sunlight.

Store at room temperature (about 10-25°C).

Available in 10 and 30 piece packs.

Description of Symbols Used on the Packaging:

	Keep away from sunlight		Do not reuse
	Do not use if package is damaged		Do not re-sterilize
	See instruction for use/warning		Date of manufacture
	Temperature limitation		Expiry date: Month/Year
	Manufacturer		Sterilization method: ethylene oxide gas
	Keep away from moisture		Lot Number